

GOWN REMOVAL GUIDE

Follow these steps to safely and efficiently remove HALYARD* Surgical Gowns after use.



Step 1: Pop the outside tie or untie with gloved hands.



Step 2: Pop the inside tie by grabbing the gown in a clump at the waist and pulling.



Step 3: Grasp gown at both sides at the level of the ties.



Step 4: To separate hook & loop closure at the neck, roll the shoulders forward while simultaneously lifting the elbows.†



Step 5: Thrust the fabric outward from the hips.



Step 6: Proceed to remove and dispose of the gown as usual.

†NOTE: The hook & loop may grab more tightly when gown is pulled across the shoulders. Steps 3 & 4 ensure release by separating front and back.

For more information for clinicians and product code ordering information, please visit:
www.halyardhealth.com

Call 1-844-HALYARD (1-844-425-9273)
in the United States and Canada.