

DONNING GUIDE

HALYARD* Surgical Gowns are "book" folded for quick and easy sterile presentation. Just follow these steps to help maintain sterility.



Step 1: Pick up the gown grasping at the single fold below the neck.



Step 2: Place the fingers of each hand under the side flaps of the arm holes.



Step 3: Open gown laterally.



Step 4: Allow the body of the gown to fall open. Extend arms into the sleeves.



Step 5: Have Circulator fasten inside rear waist ties and then fasten neck closure.



Step 6: Grasp top portion of the transfer card with the right hand and remove the left side tie with the left hand.



Step 7: Hand the bottom portion of the transfer card to assistant.



Step 8: Turn to the left, pulling the waist tie from the transfer card held by assistant. Tie the waist ties at the left side of the gown.

For more information for clinicians and product code ordering information, please visit:
www.halyardhealth.com

Call 1-844-HALYARD (1-844-425-9273)
in the United States and Canada.