

Femoral Angiography/IJ
Access Combination Drape

DRAPING GUIDE

This combination drape has multiple fenestrations for use in both femoral and IJ line placements.



Step 1: Locate body stamp on folded drape and orient to the patient's femoral or IJ access point. Remove release liner paper for one or both fenestrations.




Step 2: Align drape with the selected access points and lower directly onto the insertion site. (IJ access point shown here)



Step 3: Unfold drape, first laterally, then pull down to cover patient's feet.



Step 4: Continue unfolding drape to cover patient's head.

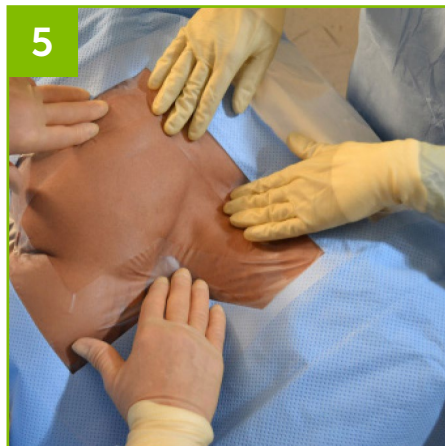

Step 5: Firmly seal edges of fenestration to patient's skin to complete draping.



Step 6: This drape has clear windows on both sides for viewing and handling controls.

Featured drape (29225/29226NS)

For more information for clinicians and product code ordering information, please visit:
www.halyardhealth.com

Call 1-844-HALYARD (1-844-425-9273)
in the United States and Canada.