

Pacemaker Drape with Fenestrated Incise

DRAPING GUIDE

This drape features two clear fenestrated incise windows with adhesive fenestrations to aid correct placement.



Step 1: Orient drape with patient and remove fenestration release paper for one or both fenestrations.



Step 2: Position drape so the chosen fenestration(s) is properly aligned with the subclavian access points.



Step 3: Unfold drape, first laterally, then toward the head.



Step 4: Unfold toward feet until patient is completely covered.



Step 5: Firmly seal edges of fenestration to patient's skin to complete draping.



Step 6: Insert surgical lines and tubing through the fabric tube holders and absorbent pouches (located on each side) to complete draping.



Step 6: This drape has clear windows on both sides for viewing and handling controls.

Featured drape (29229/29231NS)

For more information for clinicians and product code ordering information, please visit:

www.halyardhealth.com

Call 1-844-HALYARD (1-844-425-9273) in the United States and Canada.