


PICC Full Body Drape

DRAPING GUIDE

The HALYARD* Peripherally Inserted Central Catheter (PICC) Drape provides full body coverage for central line placement in the brachial area.



Step 1: After the arm has been prepped, place a sterile drape under the arm. Align the head of the body stamp on the drape with the head of the patient.

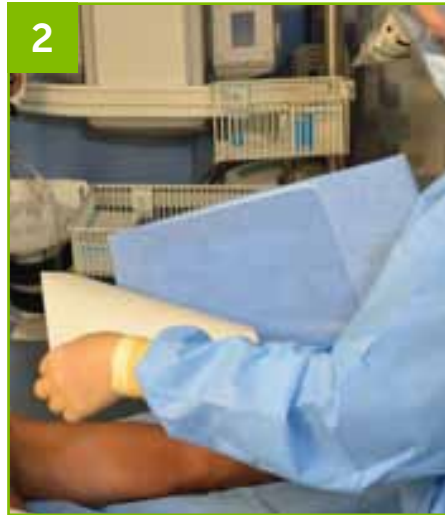

Step 2: Locate the left /right markings on the adhesive liner and align with the prepped arm. Peel away the adhesive liner.



Step 3: Center and place the fenestrated area of the drape over the insertion site.



Step 4: Following the directional arrows, unfold the drape laterally to both sides.



Step 5: Continue to follow arrows and unfold towards the head, then pull the drape towards the foot of the bed. (for recommended full body coverage.)



Step 6: Remove the blue liner sheet from the fenestration area. Press firmly with fingers around the fenestration to seal the adhesive to the skin.

See following page for perforated options

DRAPING GUIDE

FOR SIDE PERFORATION REMOVAL



Step 7: Once the line has been inserted and transparent dressing has been applied, beginning at the inside edge of the perforated incise, separate perforation and continue to the edge of the drape to remove.

FOR TOP PERFORATION USE



Step 8: Hold top of drape where the stamp is located. Pull the right side away and separate.

For more information for clinicians and product code ordering information, please visit:

www.halyardhealth.com

Call 1-844-HALYARD (1-844-425-9273) in the United States and Canada.