

# WRAPPING

**Stronger. 64% more durable.<sup>2</sup>**  
**Faster. 59% faster wrapping.<sup>1</sup>**



**1**

- Position tray adjacent to reference line and on top of blue reinforcement zones.

*If using a tray liner, place underneath the instrument tray before wrapping.*



**2**

- Fold first layer over tray and cover completely.



**3**

- Gather side.



**4**

- Fold side up and secure.
- Repeat steps 3-4 on other side.



**5**

- Gather top at both sides and fold inward.



**6**

- Fold over to cover tray.



**7**

- Secure with indicator tape and label.



**8**

- This is what the final tray should look like.

**Halyard Health Representative**

---

For more information for clinicians and product code ordering information, please visit:

[www.halyardhealth.com](http://www.halyardhealth.com)

[www.smart-fold.com](http://www.smart-fold.com)

Call 1-844-HALYARD (1-844-425-9273) in the United States and Canada.

# UNWRAPPING

**Faster. 55% faster unwrapping.<sup>1</sup>**  
**Faster. 86% faster inspection.<sup>1</sup>**  
**Stronger. 64% more durable.<sup>2</sup>**



**1**

- Break closure tape.
- Unfold top layer.



**2**

- Grab side tabs and unfold to sides.



**3**

- Gently lift tabs from fabric and pull towards you.



**4**

- Collect clean used wrap for recycling.



**Halyard Health Representative**

---

For more information for clinicians and product code ordering information, please visit:

[www.halyardhealth.com](http://www.halyardhealth.com)

[www.smart-fold.com](http://www.smart-fold.com)

Call 1-844-HALYARD (1-844-425-9273) in the United States and Canada.