

← Regular

Small →

← Small


Regular →

Place Nose Here




Sizing tool is specific to HALYARD* DUCKBILL* Surgical N95 Respirators and does not replace OSHA required product fit testing.

Instructions:
Print on 8.5 x 11 inch paper. Do not scale.

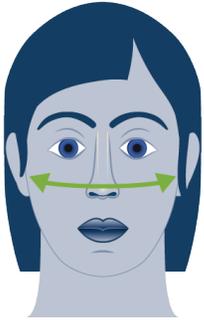

Measure Width:

1. Take measurement for width across the face between the outside of the cheek bones.
2. Compare your measurements to the data table below to find a recommended size.

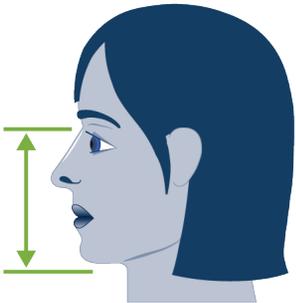

Measure Length:

1. Take measurement for length from the bridge of the nose, between the eyes, to the bottom of the chin.
2. Compare your measurements to the table to determine a recommended size.

Measurement

Recommendation

	Length	Width
Small Size	< 4 ¹ / ₂ in < 115 mm	< 5 ³ / ₈ in < 135 mm
Regular Size	> 4 ¹ / ₂ in > 115 mm	> 5 ³ / ₈ in > 135 mm

Name: _____


Date: _____

Mask Size (check):


Small

Regular

*Registered Trademark or Trademark of O&M Halyard or its affiliates. © 2021. All rights reserved.
COPY-03929

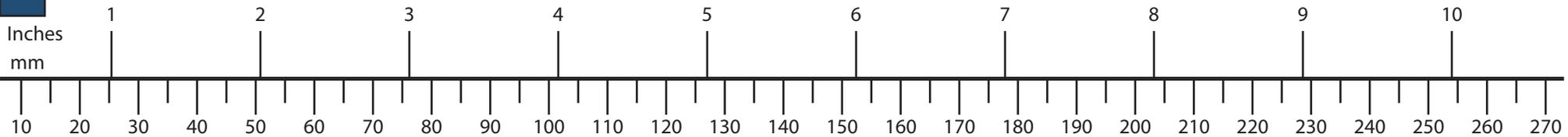

Regular
Small
Length
Small
Regular

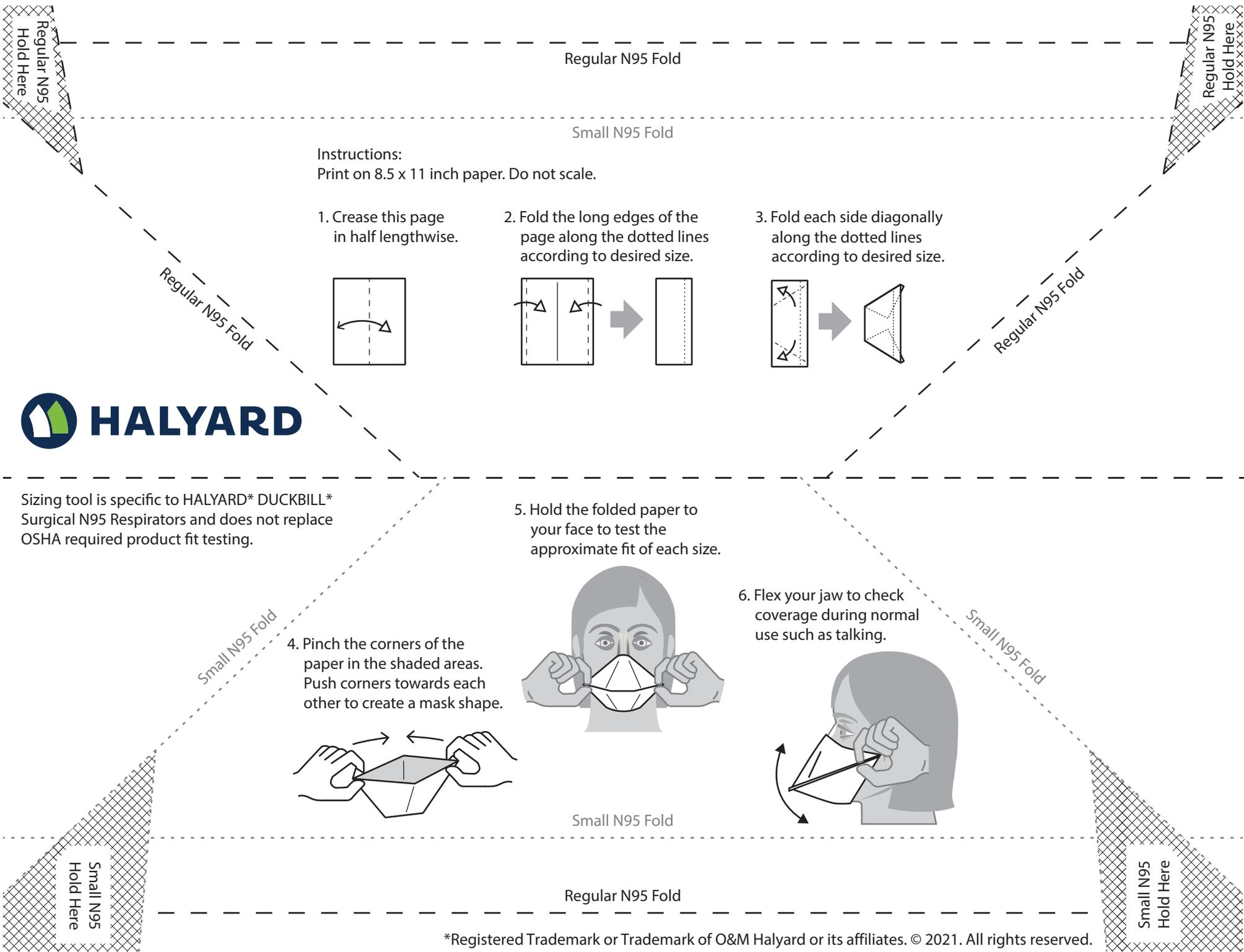


Regular N95 Fold


Small N95 Fold

Instructions:
Print on 8.5 x 11 inch paper. Do not scale.


1. Crease this page in half lengthwise.

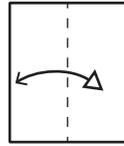

2. Fold the long edges of the page along the dotted lines according to desired size.

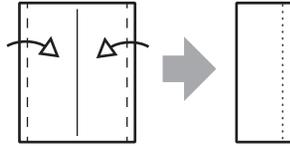


3. Fold each side diagonally along the dotted lines according to desired size.

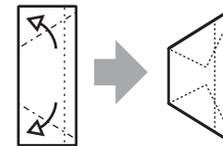


Sizing tool is specific to HALYARD* DUCKBILL* Surgical N95 Respirators and does not replace OSHA required product fit testing.

5. Hold the folded paper to your face to test the approximate fit of each size.

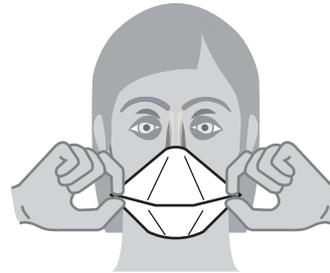


Small N95 Fold

6. Flex your jaw to check coverage during normal use such as talking.

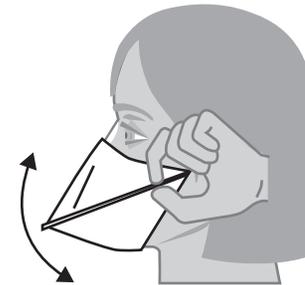

Small N95 Fold

4. Pinch the corners of the paper in the shaded areas. Push corners towards each other to create a mask shape.

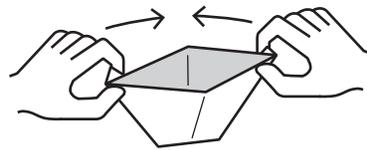

Regular N95 Fold