



WRAP GRADE RECOMMENDATIONS*

Wrap Grade	For:	Tray Weight (lbs.)
H100	Light linen packs, linen gowns, towel packs, single basins and other lighter wrapping needs.	0 - 3
H200	Linen towels, packs and gowns, light basin sets and single instruments.	3 - 6
H300	Basin sets, instrument sets, procedure trays and large linen packs.	6 - 9
H400	Most basin sets, instrument sets, procedure trays, large linen packs, and drills.	9 - 13
H500	Heavy instrument sets, retractors, and applications where frequent handling occurs.	13 - 17
H600	Extra heavy instrument sets such as Orthopedics and Cardiovascular	17 - 25
H650	Extra heavy instrument sets such as Orthopedics and Cardiovascular Also used for loaner trays.	17 - 25

WRAP TRAY SIZES

Tray Sizes	Approximate Wrap Size	Tray Sizes	Approximate Wrap Size
6" x 10" x 1 1/2"	24" x 24"	11" x 16" x 3 1/2"	45" x 45"
6" x 10" x 2"	"	11" x 16" x 4"	"
6" x 10" x 2 1/2"	"	11" x 16" x 4 1/2"	"
		11" x 16" x 5"	"
6" x 10" x 3"	30" x 30"	11" x 16" x 5 1/2"	"
6" x 10" x 3 1/2"	"		
6" x 10" x 4"	"	7" x 20" x 3 1/2"	48" x 48"
6" x 10" x 4 1/2"	"	7" x 20" x 5"	"
6" x 10" x 5"	"	11" x 22" x 1 1/2"	"
6" x 10" x 5 1/2"	"	11" x 22" x 2"	"
11" x 22" x 2 1/2"	"		
11" x 11" x 1 1/2"	36" x 36"		
11" x 11" x 2"	"	11" x 22" x 3"	54" x 54"
11" x 11" x 2 1/2"	"	11" x 22" x 3 1/2"	"
11" x 11" x 3"	"	11" x 22" x 4"	"
11" x 11" x 3 1/2"	"	11" x 22" x 4 1/2"	"
11" x 11" x 4"	"	11" x 22" x 5"	"
11" x 11" x 4 1/2"	"	11" x 22" x 5 1/2"	"
11" x 11" x 5"	"		
11" x 11" x 5 1/2"	"	11" x 22" x 6 1/2"	60" x 60"
11" x 11" x 6"	"		
11" x 16" x 1 1/2"	40" x 40"	13" x 15" x 4"	"
11" x 11" x 2"	"	14" x 24" x 4 1/2"	"
11" x 11" x 2 1/2"	"		
11" x 11" x 3"	"	11" x 33" x 3 1/2"	54" x 72"
11" x 11" x 5"	"		

* Individual results may differ due to factors such as variations in handling practices, wrapping technique, folding methods, or grade standardization. Results may also differ due to the use of irregularly shaped instrument trays, which may put added stress on the wrap. Each health care facility should determine for itself which wrap grade is most appropriate for each intended use.