

PURPLE MEANS PROTECTION

HALYARD* PURPLE NITRILE* Exam Gloves are heavy-weight nitrile gloves. They provide excellent barrier protection from chemicals, foreign substances and bodily fluids.




Trusted Performance

Excellent choice for healthcare professionals, EMS, and ER staff who need proven barrier protection from bacteria, blood-borne viruses and chemicals during procedures where risk of fluid exposure is moderate to high.



Tested and Approved

PURPLE NITRILE* Exam Gloves have been tested and approved for use with **29 chemo drugs**. Halyard cares about keeping YOU protected. That's why we list approved chemo drugs on our product packaging.

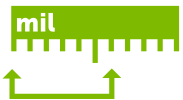

6 Mil at Fingertips

PURPLE NITRILE* Exam Gloves are thicker than all the other Halyard 9.5" exam gloves, providing increased worry-free protection.



Sterile Options Available

The individually wrapped sterile singles and pairs are ideal for sterile procedures.



PURPLE NITRILE*

Exam Gloves

ARE YOU PROTECTED? KNOW FOR SURE.



29 Chemo Drugs

PURPLE NITRILE* Non-Sterile Exam Gloves are tested for use with Chemotherapy Drugs: Per ASTM D6978

The following chemotherapy drugs and concentration had NO breakthrough detected up to 240 minutes:

Bleomycin Sulfate (15 mg/ml)	Dacarbazine (10 mg/ml)	Fludarabine (25 mg/ml)	Mechlorethamine HCL (1 mg/ml)	Rituximab (10 mg/ml)
Busulfan (6 mg/ml)	Daunorubicin HCL (5 mg/ml)	Fluorouracil (50 mg/ml)	Melphalan (5 mg/ml)	Trisenox (0.1 mg/ml)
Carboplatin (10 mg/ml)	Docetaxel (10 mg/ml)	Gemcitabine HCl (38 mg/ml)	Methotrexate (25 mg/ml)	Vincristine Sulfate (1 mg/ml)
Cisplatin (1 mg/ml)	Doxorubicin HCL (2 mg/ml)	Idarubicin HCl (1 mg/ml)	Mitomycin-C (0.5 mg/ml)	
Cyclophosphamide (20 mg/ml)	Epirubicin (Ellence) (2 mg/ml)	Ifosfamide (50 mg/ml)	Mitoxantrone (2 mg/ml)	
Cytarabine HCL (100 mg/ml)	Etoposide (20 mg/ml)	Irinotecan (20 mg/ml)	Paclitaxel (6 mg/ml)	

The following chemotherapy drugs and concentration showed breakthrough detected in less than 30 minutes:

Carmustine (3.3 mg/ml): 1.8 minutes **ThioTEPA** (10.0 mg/ml): 1.6 minutes

Warning: Not for Use with: Carmustine, ThioTEPA

Caution: The testing conditions used are intended to approximate the worst case conditions for clinical use. Testing was conducted on single layer glove material. It is the users' responsibility to determine the applicability of these gloves for their intended use with chemotherapy drugs.

Standard Practice for Assessment of Resistance of Medical Gloves to Permeation by Chemotherapy Drugs



41 Chemicals

PURPLE NITRILE* Exam Gloves provide protection against the following chemicals: Per ASTM F739

The following chemicals had breakthrough times as follows:

Adhesive Tape Remover Pads	>132.7 min	Formaldehyde (10%)	> 480 min	Monse's Solution	> 480 min
Alcoholic Bouin's Fixative	140 min	Formalin Alcohol		OxyCide Concentrate	146.7 min
Alcoholic Eosin	21.3 min	- Tested for Formaldehyde	22.0 min	OxyCide Ready to Use	> 480 min
Bleach	> 480 min	- Tested for Reagent Alcohol	> 480 min	Permaslip Mounting Medium	
Bouin's Fixative	> 480 min	Formalin, 10% Buffered		& Liquid Cover Slip	0 min
B-Plus Fixative	> 480 min	- Tested for Formaldehyde	> 480 min	Povidone Iodine (10%)	> 480 min
CarNoy's Fluid	0 min	- Tested for Methanol	> 480 min	Quaternary Cleaner	> 480 min
Chlorhexidine Gluconate (4%)	> 480 min	Giemsa Stain Solution	13.3 min	Rapid Bone Decalcifier	> 480 min
Cidex 14-Day	> 480 min	Glutaraldehyde, 4%	> 480 min	Reagent Alcohol	13.3 min
Cidex OPA	213.3 min	Haematoxylin Stain (Harris Solution)	40.0 min	Resert XL HLD	266.7 min
Decalcifier I	> 480 min	Hydrochloric Acid (35.5%)	214.3 min	Sodium Hydroxide (40%)	> 480 min
EM Fixative	> 480 min	Hydrogen Peroxide (3%)	> 480 min	Sodium Hypochlorite (10 - 13%)	> 480 min
Eosin Stain	80.0 min	Isopropyl Alcohol (70%)	40.3 min	Trichloroacetic Acid (10%)	> 480 min
Ethidium Bromide (0.4%)	> 480 min	Methanol	8 min	Xylene (99%)	0 min
Ethyl Alcohol (95%)	16.0 min	Methyl Methacrylate (Bone Cement)	0 min		

Standard Test Method for Permeation of Liquids and Gases through Protective Clothing materials under Conditions of Continuous Contact

PURPLE NITRILE*

Exam Gloves

Product Specifications

Gague Thickness Measurements	MM	MIL
Middle Finger:	.15	5.9
Palm:	.12	4.7
Cuff:	.09	3.5
Average Length	242mm	(9.5")

Physical Properties

	After Aging
Tensile Strength:	21 MPa
Ultimate Elongation:	500%

Quality Standards

Meets or exceeds ASTM D6319 - Standard Specification for NITRILE Examination Glove for Medical Applications.

Biocompatible per ISO 10993 studies for systemic toxicity, irritation and sensitization endpoints.

Passes Viral Penetration Testing - ASTM F1671 - 07 Standard Test Method for Resistance of Materials Used in Protective Clothing to Penetration by Blood-Borne Pathogens Using Phi-X174 Bacteriophage Penetration as a Test System.

Design Enhancements



Not formulated with Natural Rubber Latex – primary material is acrylonitrile butadiene, reducing the potential for glove-associated Type I allergic reactions to natural rubber latex proteins.



Powder-free, minimizing the potential for powder-related complications, such as irritant contact dermatitis.

All physical properties are target measurements; target MIL measurements may differ due to rounding.

HALYARD* PURPLE NITRILE* Exam Gloves

PURPLE NITRILE* Exam Gloves have an AQL of 1.0 for critical defects. This means that PURPLE NITRILE* Exam Gloves are significantly less likely to have critical defects than exam gloves that just meet the ASTM 2.5 AQL requirement.

Non-Sterile

Item	Description	Size	Packaging
55080	PURPLE NITRILE* Powder-Free Exam Gloves	X-Small	100 gloves/box 10 boxes/case
55081	PURPLE NITRILE* Powder-Free Exam Gloves	Small	100 gloves/box 10 boxes/case
55082	PURPLE NITRILE* Powder-Free Exam Gloves	Medium	100 gloves/box 10 boxes/case
55083	PURPLE NITRILE* Powder-Free Exam Gloves	Large	100 gloves/box 10 boxes/case
55084	PURPLE NITRILE* Powder-Free Exam Gloves	X-Large	90 gloves/box 10 boxes/case

Sterile

Item	Description	Size	Packaging
52101	PURPLE NITRILE* Sterile Single Exam Gloves	Small	100 gloves/box 4 boxes/case
52102	PURPLE NITRILE* Sterile Single Exam Gloves	Medium	100 gloves/box 4 boxes/case
52103	PURPLE NITRILE* Sterile Single Exam Gloves	Large	100 gloves/box 4 boxes/case
55091	PURPLE NITRILE* Sterile Pairs Exam Gloves	Small	50 pairs/box 4 boxes/case
55092	PURPLE NITRILE* Sterile Pairs Exam Gloves	Medium	50 pairs/box 4 boxes/case
55093	PURPLE NITRILE* Sterile Pairs Exam Gloves	Large	50 pairs/box 4 boxes/case



PURPLE EXAM GLOVE SERIES

Simplifying Your Choices

With so many glove options it can be overwhelming to choose the right glove for each situation you encounter. That is why Halyard has made it easy to choose gloves by providing this chart to explain the differences in our HALYARD* PURPLE Nitrile* Exam Glove Series. With varying cuff lengths, sterility, and chemo drug permeations, Halyard has a glove for any situation.

NON - STERILE



29 CHEMO DRUGS TESTED



52 CHEMO DRUGS TESTED

9.5" HALYARD* PURPLE NITRILE* Exam Gloves

Item	Description	Size	Packaging
55080	PURPLE NITRILE* Powder-Free Exam Gloves	X-Small	100 gloves/box 10 boxes/case
55081	PURPLE NITRILE* Powder-Free Exam Gloves	Small	100 gloves/box 10 boxes/case
55082	PURPLE NITRILE* Powder-Free Exam Gloves	Medium	100 gloves/box 10 boxes/case
55083	PURPLE NITRILE* Powder-Free Exam Gloves	Large	100 gloves/box 10 boxes/case
55084	PURPLE NITRILE* Powder-Free Exam Gloves	X-Large	90 gloves/box 10 boxes/case

12" HALYARD* PURPLE NITRILE-XTRA* Exam Gloves

Item	Description	Size	Packaging
55090	PURPLE NITRILE-XTRA* Powder-Free Exam Gloves	X-Small	50 gloves/box 10 boxes/case
50601	PURPLE NITRILE-XTRA* Powder-Free Exam Gloves	Small	50 gloves/box 10 boxes/case
50602	PURPLE NITRILE-XTRA* Powder-Free Exam Gloves	Medium	50 gloves/box 10 boxes/case
50603	PURPLE NITRILE-XTRA* Powder-Free Exam Gloves	Large	50 gloves/box 10 boxes/case
50604	PURPLE NITRILE-XTRA* Powder-Free Exam Gloves	X-Large	50 gloves/box 10 boxes/case

STERILE



NO CHEMO CLAIMS AT THIS TIME



30† CHEMO DRUGS TESTED

† 52 coming soon! Additional testing in progress

9.5" HALYARD* PURPLE NITRILE* Exam Gloves

	Item	Description	Size	Packaging
SINGLES	52101	PURPLE NITRILE* Sterile Single Exam Gloves	Small	100 gloves/box 4 boxes/case
	52102	PURPLE NITRILE* Sterile Single Exam Gloves	Medium	100 gloves/box 4 boxes/case
	52103	PURPLE NITRILE* Sterile Single Exam Gloves	Large	100 gloves/box 4 boxes/case
PAIRS	55091	PURPLE NITRILE* Sterile Pairs Exam Gloves	Small	50 pairs/box 4 boxes/case
	55092	PURPLE NITRILE* Sterile Pairs Exam Gloves	Medium	50 pairs/box 4 boxes/case
	55093	PURPLE NITRILE* Sterile Pairs Exam Gloves	Large	50 pairs/box 4 boxes/case

12" HALYARD* PURPLE NITRILE-XTRA* Exam Gloves

	Item	Description	Size	Packaging
PAIRS	14259	PURPLE NITRILE-XTRA* Sterile Pairs Exam Gloves	X-Small	50 pairs/box 4 boxes/case
	14260	PURPLE NITRILE-XTRA* Sterile Pairs Exam Gloves	Small	50 pairs/box 4 boxes/case
	14261	PURPLE NITRILE-XTRA* Sterile Pairs Exam Gloves	Medium	50 pairs/box 4 boxes/case
	14262	PURPLE NITRILE-XTRA* Sterile Pairs Exam Gloves	Large	50 pairs/box 4 boxes/case
	14263	PURPLE NITRILE-XTRA* Sterile Pairs Exam Gloves	X-Large	50 pairs/box 4 boxes/case